



PX4 PAPR Kit Components

- PX4 Air Unit with Filter
- Air Hose
- Belt and Battery
- Battery Charging Station
- Pre-Filters
- Reusable Hood
- Reusable Helmet
- Flow Tester
- Air Flow and Alarm Logs

*If equipment is malfunctioning
or broken, please call
Biomedical Engineering*

How to Clean Unit After Use

1. Arrange for another staff member (PPE Buddy) to wait outside of the patient room to assist with doffing.
 - a. PPE Buddy should wear a white apron and gloves.
 - b. A designated table/countertop space may be used in lieu of a PPE Buddy.
2. While still wearing the PAPR, an initial wipe-down of equipment with a hospital-approved disinfectant will be performed inside of the room.
3. At the threshold of the room, PAPR will then be taken off and handed to PPE Buddy or placed on designated table/countertop. ****Do not turn PAPR off****
 - a. Unclip belt
 - b. Remove PX4 Air unit from around waist
 - c. Remove hood
4. Doff additional PPE at threshold of the room.
5. Perform hand hygiene.
6. Turn off PAPR.
7. Perform a second, thorough wipe down of PAPR (PX4 Air Unit, Belt, Hood (inside and outside)).
8. Place all PAPR equipment back in designated bin.
9. Return to a centralized location for redistribution.